

RELEASE NOTE

BUILD ID: 2E4B0696151E00518FA6C8820A8769CA62BDD8
OLEA SPHERE V2.2SP7 - ONLY DISTRIBUTED BY VITAL



Olea Medical® SUPPORT
93, avenue des Sorbiers - 13600 La Ciotat - FRANCE
TEL +33 (0)4 42 71 72 78 - FAX +33 (0)4 42 71 24 27
support@olea-medical.com

BUG FIXES:

- Configuration menu: permeability contrast agent choice is now taken into account allowing permeability data computation.
- Fiber tracking application module: ROI deletion does not cause any loss of 2D views.

KNOWN ISSUES:

- Configuration menu and Analysis plug-in: VOI category color is not synchronized between the configuration and the volume segmentation tab.
- Analysis plug-in - VOI creation: when using the VOI options without hitting ENTER, the VOI is displayed but cannot be saved.
- Analysis plug-in: when loading an image in an empty cell, the image is not correctly positioned. It is needed to type "f" to reposition it.
- Analysis plug-in: brightness and contrast are not the same by default between conventional series and Misc series.